
Depaepe

HD Max Ampli

Mode d'emploi/User manual
Ref : 18032008

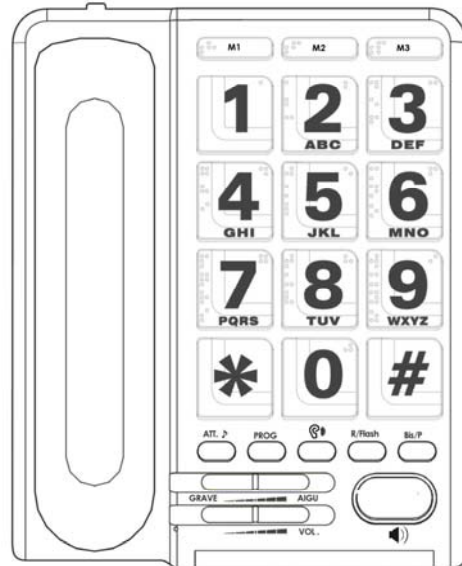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
 **CAUTION** : Before using this phone, and, in order to prevent from potential acoustic shocks all users who are not hearing impaired, make sure that the “Amplify switch” located under the phone is on the “MANUAL” position.
See “Amplify switch” section for more details.

Connecting and installing

Plug one end of the twisted cord into the socket provided on the end of the handset, and the other end to the socket on the left side of the phone.
Connect the straight telephone line cord to the socket at the rear of the phone and the other end to the telephone outlet. Connect the AC adaptor to the back of the telephone and to the electrical outlet. A red led on the front panel will light to confirm that power is on.

Install batteries

 **CAUTION** : Before opening the battery compartment cover, disconnect from the line and from the AC adaptor to prevent from potential electrical shock. Use a tool like a small screwdriver or a pencil to push the battery compartment cover open.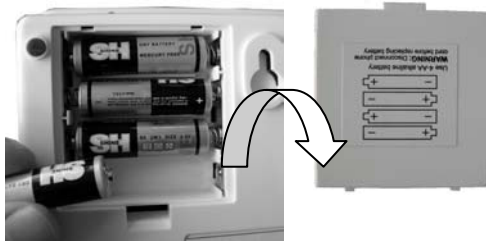

Open with a tool
(Screwdriver for instance)
Batteries not included


If you wish to save numbers stored into memories and allow some amplifying in the handset in case of AC power shortage, install 4 AA batteries (Not included). Make sure to observe the batteries polarity as displayed on the label of the battery compartment cover.
When using this telephone with battery power, a low battery indicator on the front panel will light up if the batteries are low.

Wall mounting

The HD Max Ampli can be wall mounted by following the procedure below:

- a) Slide out the clip located next to the hook switch and turn it upside down before sliding it back into place.

Note – the clip will locate into a recess in the handset, which will hold the handset when wall mounted.

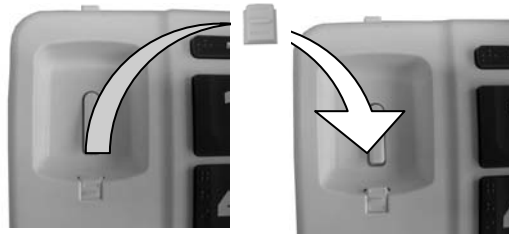- b) Fit the wall bracket (supplied with the phone) under the base of the phone, locating the tabs on the bracket into the slots in the phone.

- c) Select an area of the wall where you would like to hang your phone and check that there are no pipes or electrical wires buried in the wall in this area.
- d) Drill 2 holes and fix 2 screws at 10cm intervals, leaving the heads of the screws protruding.
- e) Align the keyhole slots in the rear of the phone with the heads of the screws and slide down slightly downwards so that the heads of screws locate in the slots. If necessary, adjust the screws so that the phone hangs flat and holds firmly on the wall.

Ringer settings

On the right side of the telephone, 2 switches allow a selection of “HIGH”, “MEDIUM” or “LOW” ringer setting and a choice 2 different melodies.

Note : The HD Max Ampli will also alert visually all entering calls with flashing leds located on the front and on the right side of the telephone.

Making or answering a call

With the handset :

- a) Pick up the handset
- b) Enter the number to dial or answer the call
- c) Hang up the handset at the end of the conversation

With the “handsfree” speaker

- a) Press “

Switching between the handset and the speaker :

While talking with the handset, you can switch to speakerphone mode by pressing “

11

REMINDER :  **CAUTION** : Before using this phone, and, in order to prevent from potential acoustic shocks all users who are not hearing impaired, make sure that the “Amplify switch” located under the phone is on the “MANUAL” position.
See “Amplify switch” section for more details.

Adjusting Receive volume and Tone

The volume and tone of the speech received through the handset can be adjusted for a better understanding by hearing impaired people. Those adjustments can be made using the 2 sliding switches located on the front panel of the telephone.

To activate those sliding control switches, the “” button must be pressed; a red led on the front panel will confirm this activation.

- Pick up the handset
- Press “” key (the red led will light)
- Adjust the sliding switches as needed.

Note : if the “amplify switch” located under the telephone is in the “ON” position, pressing “” key is not necessary.

Amplify switch

When the Amplify switch (located under the base) is in the “MANUAL” position, the receive speech boost will be canceled at the end of each conversation, when hanging up the handset.

The “MANUAL” mode is convenient when different users, hearing impaired or not, use the same HD MAX Ampli telephone; activating manually this feature, on demand, by pressing the “” key minimizes potential acoustic shocks.

However, if the HD MAX Ampli should always be used with the receive boost on, then the Amplify switch should be in the “ON”

mode ; this way, the amplification boost will always be active even after each hanging up of the handset.

Note : in the speakerphone mode, pressing “” key is not necessary, it is automatically activated, whatever the position of the amplify switch under the telephone.

BIS/P (LNR REDIAL/PAUSE)

This key allows the last number dialed to be automatically redialed (maximum capacity is 32 digits).

Pressing this key when dialing enters a timed pause in the dialing process: this may be useful when storing a number into a memory key (see “Storing a Telephone number” section).

ATT

When pressing this key during a call, the other party will be put on hold (with music while on hold). The handset can be hung up and a red led will light to alert someone is on hold. To resume the conversation, simply pick up the handset or press “” key again to resume the conversation in the “speakerphone” mode.

One touch memory keys

3 numbers can be stored into the memory keys M1, M2 and M3.

To call a number stored in one of these 3 locations, simply pick up the handset and press the memory key. Each memory can store up to 16 digits numbers.

Note : disconnecting the phone from the line will erase the numbers stored into the memory keys.

Storing a new number into a location where one number is already stored will automatically erase the previous number stored.

To erase a memory without storing a new number, simply follow the storing process (detailed here after) without entering any number.

Storing a number into a one touch memory key

- Pick up the handset
- Press PROG key
- Enter the number to store (maximum of 16 digits) and, if needed, insert a timed pause (by pressing P key) each time a tone has to be heard before continuing the dialing process (useful when dialing behind a PABX or for long distance services access for instance).
- Press PROG key again.
- Press the memory key where this number should be stored (M1, M2 or M3).
- Hang up the handset
- Repeat this operation for the other one touch memory keys.

Calling a number stored in M1,M2 or M3


- Pick up the handset
- Press M1, M2 or M3, the number stored will be dialed automatically.

R key (Flashing)

This key is used to transfer a call to another extension when the HD MAX Ampli is connected behind a PABX. Flashing time can be adjusted to 100 or 300ms, the setting switch is located on the right side of the telephone. In France, most of the PABXs will use 300 ms flashing time. In almost all other Europeans countries, 100 ms setting should be used.

Audio output

This audio output, located on the right side of the telephone can be used to connect a headset (without microphone), a speaker or to record a conversation.

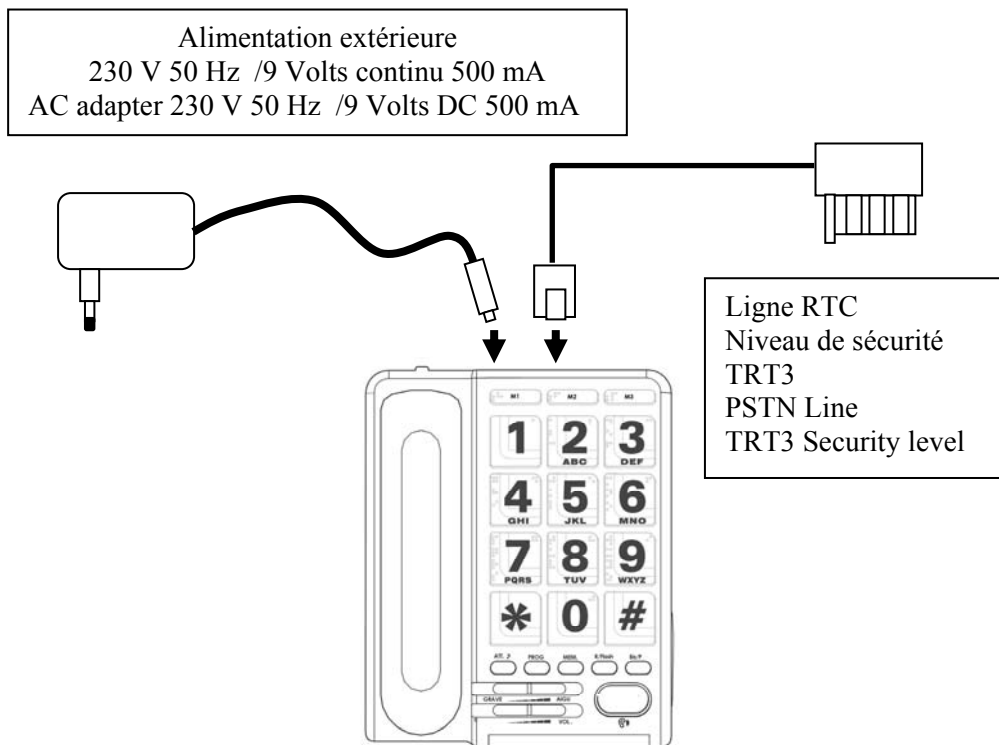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This marking indicates that this product complies with the European Union Waste Electrical and Electronic Equipment directive (WEEE), 2002/96/EC. It should not be disposed of with other household wastes at the end of its working life to prevent possible harm to the environment or human health. Please contact your supplier or your local government office to dispose of it in an approved recycling centre.